ASA DENTAL

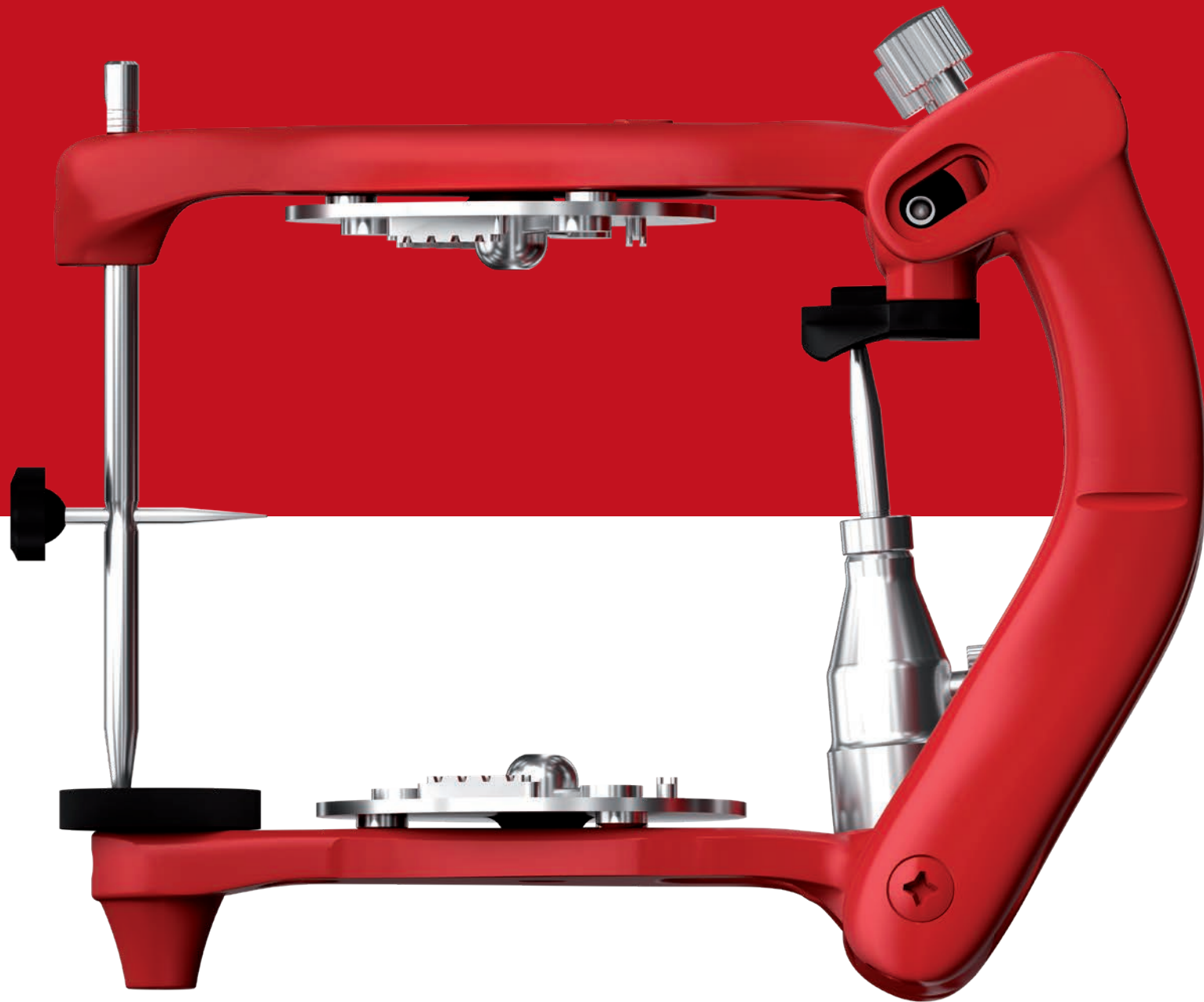
MAKE PEOPLE SMILE

MADE IN ITALY

INSTRUCTIONS FOR USE

ISTRUZIONI D'USO

5050
5050 Plus

PRODUCT DESCRIPTION

5050 and 5050Plus are non-Arcon average value articulators designed to help dental technicians meet dentists' requests for high precision, regardless of the prosthetic technique. A light, stable and ergonomic construction facilitates set-up and work on any models. Additional features guarantee an exact simulation of the condylar path with a high fitting accuracy, giving prosthesis and restorations the necessary occlusion and function throughout the full range of mandibular movements.

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INDICATIONS

The articulator is suitable for the fixation of models for lab-fabricated reconstructions. 5050 and 5050Plus allow a perfect simulation of average movement patterns of the human temporomandibular joint (TMJ).

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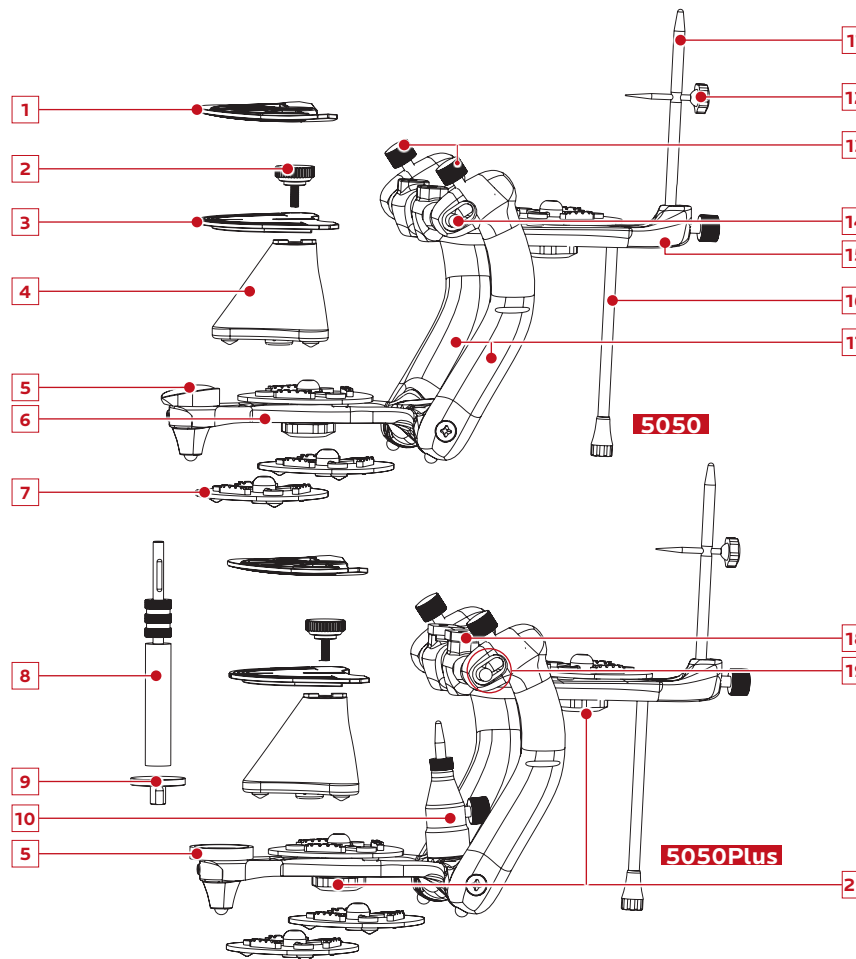
UNPACKING AND CHECKING THE CONTENTS

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5050Plus

ASSEMBLY AND INITIAL SETUP

Mount above parts depending on the selected technique or reconstruction stage.

Articulators 5050 and 5050Plus can conveniently work in 4 different positions (all these positions allow a superb overview of the model mounted in the articulator):

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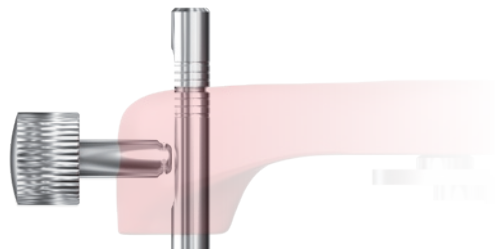
The anti-rotational system for the incisal rod ensures that positioning is always correct and accurate.

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4 elliptic references help the technician regardless the arch size and the selected starting technique.

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The back rod system allow to remove the incisal rod without affecting performances.

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COMPARISON

Technical features	5050	5050Plus
stable 60° operation for technician comfort	☑	☑
ideal for class III patients	☑	☑
compatible with any pressure polymerizers	☑	☑
anti-rotation system for incisal rod	☑	☑
template with elliptic references for lower arches models	☑	☑
occlusal table with elliptic references for upper arches models	☑	☑
magnetic rod kit	○	☑
back rod system	○	☑
4 mounting plates	☑	☑

Technical data	5050	5050Plus
intercondylar distance	110 mm	110 mm
condylar inclination fixed at	30°	30°
Bennett angle	0 - 7°	0 - 7°
incisal table inclination	10°	0°
inclined contrast table	-	24°
distance between mounting plates	87 mm	87 mm
opening	180°	180°

MAINTENANCE AND CLEANING

Monitoring and maintenance		
CHECK	PART	FREQUENCY
joint sufficiently lubricated	joint bolt	monthly/as necessary
metallic knob is dirty	metallic knob for template/occlusal table	weekly/as necessary

Cleaning*		
CHECK	FREQUENCY	MEASURE
metallic knob contaminated with wax or plaster	weekly/as necessary	rinse with warm water
joints contaminated with dust	weekly/as necessary	rinse with warm water

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105 Maxess Road, Suite 124
Melville, New York 11747 - USA

Asa Dental CHINA CO. LTD.

Room 315, Huaying Building,
Zhong Xin Avenue
Tianjin Airport Economic Area
Tianjin - China

Asa Dental S.p.A. (Branch)

Dubai South Business Center
Building C, 3rd Floor
Dubai Logistics City
PO Box 390667
Dubai - UAE

ASA DENTAL
MAKE PEOPLE SMILE

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Asa Dental USA Corp.