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## en Spray-on suspension for BEGO ceramic crucibles used with BEGO Nautilus® and Fornax® casting units

BEGO Lolipot contains a coating material specially developed for the BEGO ceramic crucibles used with *Nautilus®* and *Fornax®* casting units from BEGO.

The coating extends the useful life of the crucibles and ensures virtually residue-free casting with almost all precious metal alloys, including palladium-based alloys. It is also suitable for cobalt-chrome and nickel-chrome alloys.

Lolipot does not influence the properties of the alloy, and if its effectiveness decreases, it can be reapplied.

### 1. Shaking

- Shake Lolipot well before use.

### 2. Application

- Lightly spray interior of crucible from a distance of approx. 15 cm. Hold spray bottle upright during spraying.

or

- Apply a thin coating of Lolipot to the interior of the crucible with the brush supplied.

### 3. Drying

- After spraying, dry the crucible for 5–10 minutes at 100–250 °C (the higher the temperature, the faster the drying). Lolipot dries in seconds when applied to a hot crucible. After drying, place the crucible in the casting chamber and start the casting process – **Do not preheat the crucible!**

### Note:

To prevent the spray nozzle from getting clogged, turn the bottle upside down after use and spray 2 to 3 times until no more suspension is discharged.

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