READ CAREFULLY BEFORE USE

Primer for bonding resin to metal, zirconia and alumina

PRODUCT DESCRIPTION

SHOFU Universal Primer is a priming agent designed to treat the surfaces of metal-based, zirconia-based or alumina-based restorations to enhance bond to resin materials. SHOFU Universal Primer can be used intraorally and extraorally.

INDICATIONS FOR USE

Bonding resin to metal, zirconia or alumina restorations
> Application of opaque resin to frameworks
> Cementation of inlays, crowns and bridges with resin cement
> Bonding of denture base resin to metal base, clasp or attachment
> Bonding resin to orthodontic appliances
> Intraoral and extraoral repair of fractured restorations

PRECAUTIONS AND WARNINGS

1. Do not use this product on patients with known allergies to this material, methacrylate monomer and/or acetone.
2. Operators with known allergies to this material, methacrylate monomer and/or acetone must not use this product.
3. If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
4. Use in well ventilated areas to avoid inhalation of acetone (several appropriate ventilations per hour are recommended). If inhalation causes nausea or illness, take rest and keep warm in fresh air.
5. Avoid any source of ignition since this product is flammable.
6. Use protective gloves and glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. Medical (dental) gloves provide some protection against direct contact with this material; however some monomers may penetrate through the gloves very quickly to reach the skin. Therefore, should this material come in contact with the gloves, immediately remove them and wash hands with plenty of water.
7. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
8. If this product comes in contact with oral soft tissue, the tissue may turn whitish, however, it should resolve in 2 or 3 days. Avoid touching or brushing the area.
9. Follow the instructions for use of each material, instrument or equipment to be used in conjunction with this product.
10. Do not use this product for any purposes other than specifically outlined in these instructions for use.
11. Use this product within the expiration date indicated on the package or container.
(Example ☐ YYYY-MM-DD → Year-Month-Date of the expiration date)
12. This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

A. Application to metal

A-1. Pretreatment of the restoration surface

A-1-1. Extraoral use

Sandblast the adhesive metal surface with 50-100 µm aluminum oxide at approx. 0.3-0.5 MPa/3-5 bar. Then, perform ultrasonic cleaning and dry with clean air. Sandblast can be replaced by roughing the surface with a suitable abrasive (eg. diamond bur etc).

A-1-2. Intraoral use

Place rubber dam and avoid direct contact with oral tissue. Roughen the adhesive metal surface using a dental abrasive (eg. diamond bur etc), rinse with water and dry with clean air.

A-2. Application of SHOFU Universal Primer

Dispense adequate amount of SHOFU Universal Primer on a dish. Apply to the entire adhesive surface in one layer with a disposable brush. Leave it undisturbed for 10 sec. and dry naturally.
A-3. Bonding to resin material
Bond resin material according to the instructions for use of each product.

B. Application to zirconia and alumina

B-1. Pretreatment of the restoration surface

B-1-1. Extraoral use

Sandblast the adhesive surface of zirconia-based or alumina-based restoration with 50-100 µm aluminum oxide at approx. 0.2-0.3 MPa/2-3 bar. Then, perform ultrasonic cleaning and dry with clean air. Sandblast can be replaced by roughing the surface with a suitable abrasive (eg. diamond bur etc).

B-1-2. Intraoral use

Place rubber dam and avoid direct contact with oral tissue. Roughen the adhesive surface by using a dental abrasive (eg. diamond bur etc), rinse with water and dry with clean air.

B-2. Application of SHOFU Universal Primer

Dispense adequate amount of SHOFU Universal Primer on a dish. Apply to the entire adhesive surface in one layer with a disposable brush. Leave it undisturbed for 10 sec. and dry naturally.
B-3. Bonding to resin material
Bond resin material according to the instructions for use of each product.

(Notes)

> Do not mix with any other primers or bonding systems.
> Tightly close the cap after each use as this product contains volatile components. Use all the dispensed material promptly.
> Use saliva ejector or suction during the procedure.
> Avoid contamination of the restoration surface to be bonded. If the surface is contaminated by saliva or blood, thoroughly clean the surface with an alcohol moistened cotton ball. Then, dry and reapply the material.
> Use all the dispensed material promptly on one patient.
> Do not reuse disposable brushes to avoid cross-contamination.

COMPOSITION

Acetone, Phosphoric acid monomer, Thiocitic acid monomer

STORAGE

Store at room temperature (1-30 °C/34-86 °F). Keep away from direct sunlight and any source of ignition.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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